



How to use the ceiling hoist

Maximum weight for use is 200 kilograms.

- This hoist can be used to move and lower a person in a sling onto the change table, toilet or a wheelchair.
- If you are unsure about using the hoist please do not use this facility.
- The hoist must be charged and showing a green light before using. If a yellow light is on or there is a beeping noise do not use the hoist.
- Use the arrows on the hand control to raise or lower the spreader bar to the required height.
- Please use your own sling for health and hygiene reasons.

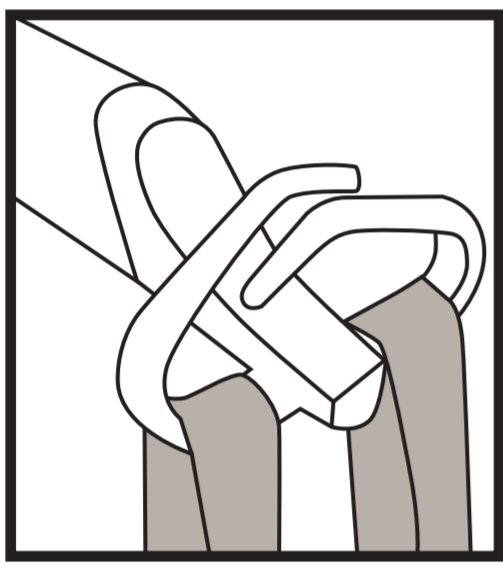

- Attach the hooks of your sling to the lifting hanger's hooks (only slings with four or six loops are compatible).

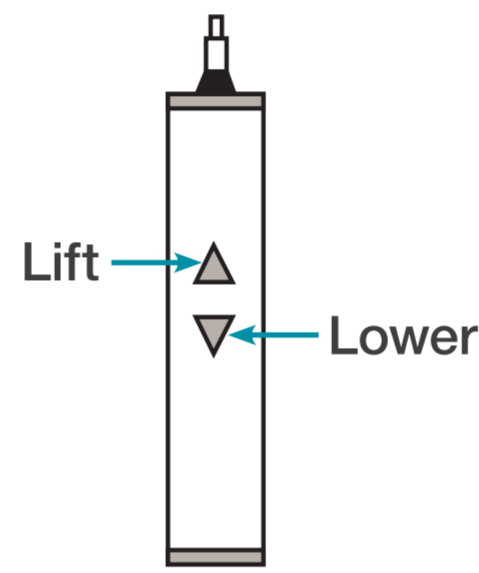

- Check sling is securely attached before lifting a person.
- Use the hand control buttons to raise the person.

- The hand control can be clipped to the spreader bar for easy access.
- Do not leave anyone unattended and suspended in the hoist.
- In the event of a power failure while someone is suspended, pull and hold the red strap continuously to lower the spreader bar.
- If the red cord is pulled accidentally the hoist will not operate until the yellow button under the hoist is gently pushed back to be flush with the casing.
- To stop in an emergency pull the red strap continuously.

For a medical emergency dial 000.

For maintenance or cleaning
1300 694 222 (business hours)
1800 637 000 (after hours)

1300 GOLDCOAST

cityofgoldcoast.com.au

CITY OF
GOLDCOAST.TM