

How to use the change table

This change table is height adjustable.

Maximum weight for use is 200 kilograms.

1. Fold down the change table from the wall into the flat position.
2. To adjust the height of the change table, use the arrow push buttons on the handset control located at the end of the table.
3. Use tear-off paper from the dispenser to cover the table before use.
4. Ensure table is cleaned with antibacterial solution provided before and after use to maintain hygiene.
5. Do not leave a person unattended on the change table.
6. End guards and side rails can be locked in place or folded away.

For a medical emergency dial 000.

For maintenance or cleaning

1300 694 222 (business hours)

1800 637 000 (after hours)

1300 GOLDCOAST

cityofgoldcoast.com.au